

USER'S GUIDE


EPILATOR EP103

Epilator EP103 is used to remove superfluous hirsute and for coagulation of tissues by means of high frequency current action. It guarantees a great aseptic, effective and almost painless treatment. Its using prevents bleeding during coagulation of the tissues.



EPILATOR EP103
nr kat. 10307

Apparatus front plate view

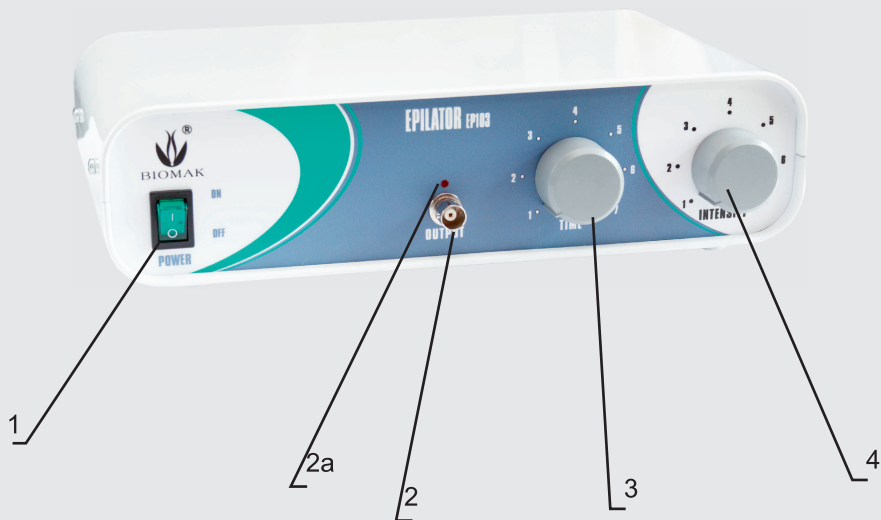

POWER	(1) – Power switch
OUTPUT	(2) – Needle grip seat
	(2a) - Operation indicator
TIME	(3) – Knob for regulating the time of the current flow
INTENSITY	(4) – 5.Knob for regulating the current intensity

Technical specification:

• Supply	230V, 50Hz
• power input	max 35W
• operation frequency	2,7 MHz
• dimensions	260x160x65mm
• weight	~2kg

Wyposażenie aparatu:

A) user manual

B) power supply cable



C) Needle fastening holder(nr kat. 00302)



D) cable connecting the apparatus with the handle



E) foot swith (nr kat. 00301)



F) UNI-PROBE needles (selected randomly)

Przygotowanie aparatu do pracy1.

1. Put the plug into the 230V, 50Hz grounded outlet.
2. Put the pedal plug into the socket on the rear panel of the device.
3. Put the plug of the grip for fixing needles into the OUTPUT(2) socket.
4. Loosen the grip for fixing needles undoing its end counterclockwise.
5. Put the needle mandrel into the grip hole.
6. Tighten the grip end clockwise.





NOTE for needles with a bayonet, e.g. Ballet, change the tip in the handle.

Przystąpienie do zabiegu1.

1. Before the treatment, clean and disinfect the skin.

2. Switch on the device with the ON/OFF(1) button.

3 Regulate the current intensity with the INTENSITY(5) knob proportionally to the kind of treatment and individual sensitivity of the patient, and regulate the time of removing hair, i.e. the time of the current flow in seconds, with the TIME(4) knob.

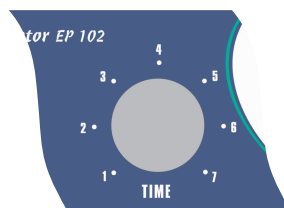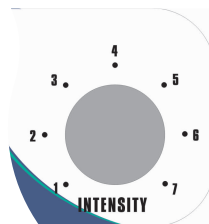

4. Put the needle into the hair channel and switch on the current, by pressing the pedal for a short time (the red lamp(2a) will switch on over the OUTPUT(2) socket, signalling that the current has been connected to the needle). The current flows for as many seconds, as the TIME knob has been set for, and then the diode(2a) switches off and a sound signal can be heard.



5. After the needle has been taken off, a hair is removed with the forceps.



In order to stop the treatment, press the pedal for a short time. The treatment time (the time of the current flow) can be freely extended, by pressing the pedal.

PRECAUTIONS

To avoid any damage of the skin:

- adjust carefully intensity of current and treatment time to the sensitivity of patient.
- Insert precisely the needle into hair canal on order to avoid superficial scald of the skin
- Supply the current only in the moment when needle reached the dermic papilla.

Do not expose the treated area to the sun within 24 hours after treatment

REMARKS CONCERNING THE SAFE USING OF THE APPARATUS

- The outlet plug should be applied to a socket with ground.
- If the apparatus falls or is hit strongly it should be sent to the producer or authorized shop for a check-up.
- If the feeder cable is damaged, it should be replaced with a new one of the same type in a service shop.
- Any attempts to fix, re-connect or change the internal system of the apparatus is forbidden.
- Opening of the apparatus or destroying the seal automatically cancels the guarantee and the producer does not resume further responsibility for the correct and safe operation of the apparatus.



BIOMAK Sp.J. 05-820 Piastów, ul. Słońskiego 13, tel.: (022) 723 70 72, tel./fax: (022) 723 69 54,
e-mail: biomak@biomak.pl www.biomak.eu